SPECTRUMDNA™

A Division of SPECTRUM SOLUTIONS, LLC

SDNA - 1000

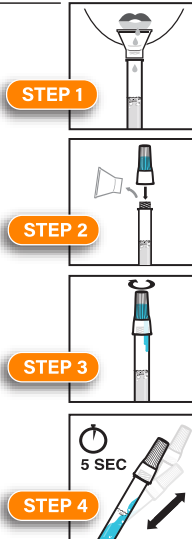
2mL Saliva Collection Device - Instructions For Use (IFU)

For use under an emergency use authorization (EUA) only.

Do NOT eat, drink, smoke, or chew gum for 30 minutes before giving your saliva sample.

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Fill the tube with saliva to the black wavy line. Fill the tube until your saliva (not including bubbles) is at or just above the wavy line. **DO NOT OVERFILL.**

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Replace the funnel with the fluid cap. Remove the funnel from the tube. Screw on the enclosed cap **TIGHTLY** to release the solution that will stabilize the DNA in your saliva.

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Firmly screw cap down to release solution and seal tube. You will know it works when the blue solution from the cap is released into the tube. Firmly tighten cap to assure the cap and tube is completely sealed.

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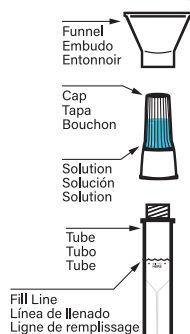
Shake the tube for at least five seconds. This will ensure your sample mixes thoroughly with the stabilizing solution.

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Made in the U.S.A.
PATENT PENDING
For In Vitro Diagnostic Use

Spectrum Solutions
12248 S Lone Peak Parkway, #106
Draper, UT 84020
spectrumsolution.com
Storage 15°C / 30°C
2019162
Quality First International OÜ
Laki 30
12915 Tallinn
Estonia
Tel: +372 61041 96

SDNA-1000 SALIVA COLLECTION DEVICE - Safety Information

*** Children under 12 should be supervised**
***Do not clean with any substance containing bleach**
***Do not use if kit is expired**
Cap choking hazard. Keep out of reach of children. Wash with water if liquid solution comes in contact with eyes or skin. DO NOT INGEST LIQUID.
This kit is designed for the collection of human saliva samples.
This self-sample collection device has not been FDA cleared or approved.
This self-sample collection device has been authorized by FDA under an EUA.
This self-sample collection device has been authorized only for the collection and maintenance of saliva specimens as an aid in detection of nucleic acid from SARS-CoV-2, not for any other viruses or pathogens.
This self-sample collection device is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostic tests for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Federal Food, Drug, and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the authorization is terminated or revoked sooner.

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Label Legend:

- Consult instructions for use
- Use by date (Collect saliva by)
- In vitro diagnostic medical device
- Catalog number
- Caution, consult instructions for use
- Temperature limit
- Authorized representative
- Manufacturer
- Batch code
- Prescription Only

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